

# DualForce™

## Sectional Matrix System




### Tighter Contacts. Better Seal.

DualForce™ is a complete matrix system designed for use with most Class II composite restorations.

#### DualForce Rings


Featuring unique dual nickel titanium wires and independent interproximal separators, **DualForce Rings** provide significant and balanced separation force and an excellent matrix/tooth seal.

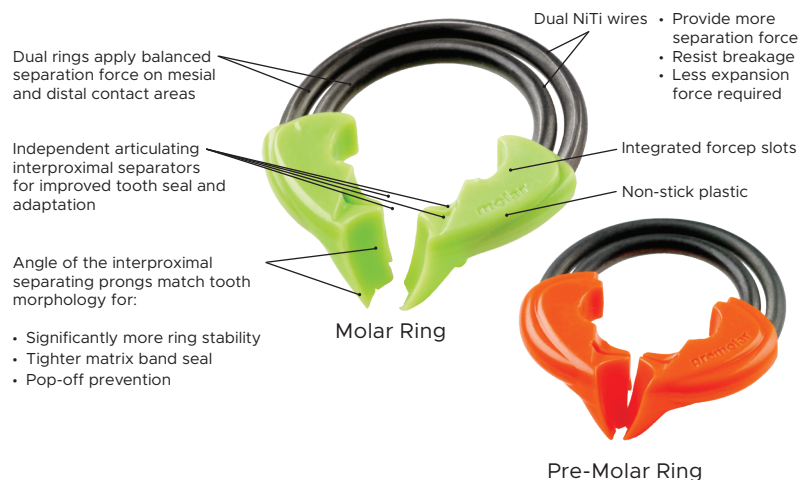

- Provide more separation force
- Resist breakage
- Less expansion force required
- Significantly more ring stability
- Tighter matrix band seal
- Pop-off prevention

#### DualForce Active-Wedges™


Designed to enhance separation force, **DualForce Active-Wedges™** unique collapsing tip re-opens after placement, locking the wedge in place, and combined with a natural contour ensures a complete marginal seal to prevent overhangs.

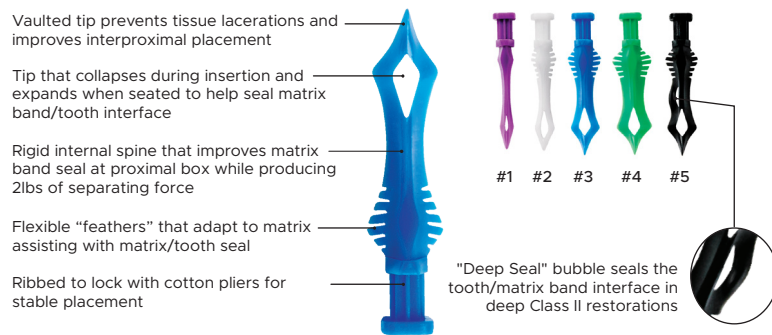

- Improves interproximal placement
- Helps seal matrix band/tooth interface
- Improves matrix band seal at proximal box while producing 2lbs of separating force
- Assisting with matrix/tooth seal
- For stable placement
- "Deep Seal" bubble seals the tooth/matrix band interface in deep Class II restorations

#### DualForce Ultra-Wrap™ Matrix Bands

**DualForce Ultra-Wrap™ Matrix Bands** are designed to wrap the tooth snugly to reduce flash and finishing time, while a proximal contour and curved marginal ridge ensures a natural contact position and shape is created every time.

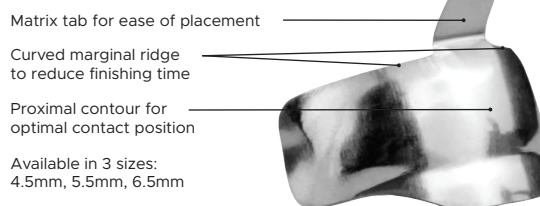

Available in 3 sizes:  
4.5mm, 5.5mm, 6.5mm



“The DualForce Sectional Matrix System provides tight contacts and anatomical contours.”

Nate Lawson, DMD, MA, PhD



Request a digital demonstration for you and your team  
[dualforcematrix.com/request-a-demo](http://dualforcematrix.com/request-a-demo)



**1** With rubber dam in place, pre-op view of wide Class II on first molar requiring replacement.



**2** Ultra-Wrap™ Matrix Band in place extending well beyond the proximal line angles. Ultra-Wrap is made from .0015" semi-hard stainless steel making it more resistant to crimping during placement.



**3** The Active-Wedge™ tip collapses during insertion for easy placement, then re-expands when fully seated, regaining full contour for an improved matrix band seal.



**4** Once seated, the DualForce™ Sectional Matrix System provides a complete gingival seal with a tight vertical seal of the matrix band along with a strong and stable separation force, resulting in less chance of composite flash.



**5** First increment of Evanesce™ restorative material adapted to include Ultra-Wrap's pre-contoured occlusal lip, establishing an ideal rolling marginal ridge.



**6** Evanesce composite adapts to the internal surfaces of the preparation with excellent packability.



**7** Evanesce is easily sculpted with no slumping, efficiently and effectively establishing the ideal occlusal anatomy before the final curing.



**8** Diamond-impregnated A.S.A.P.® Pre-polisher initiates the 2-step polishing sequence in just 10-15 seconds.



**9** The A.S.A.P. Pre-polisher gently diminishes small surface defects without affecting anatomy.



**10** After just 10-15 seconds of polishing with the A.S.A.P. Final High Shine Polisher, the final esthetic restoration with an optimal proximal contact.

**Instant Replay** / In a rush? Watch this quick technique tip! **POWERED BY Hands-OnLine LIVE**

Scan to watch

**DualForce™ Matrix Band and Active-Wedge Placement**  
with Marc Geissberger, DDS, MA

**DualForce™ Ring Placement**  
with Marc Geissberger, DDS, MA



**STARTER KIT WITH ULTRA-WRAP BANDS (209021)**

- Contains: 1 x Pre-Molar Ring (orange), 1 x Molar Ring (green), 25 x 4.5mm Ultra-Wrap Matrix Bands, 25 x 5.5mm Ultra-Wrap Matrix Bands, 25 x 6.5mm Ultra-Wrap Matrix Bands, 20 x Active-Wedge #1 (Purple), 20 x Active-Wedge #2 (White), 20 x Active-Wedge #3 (Blue), 20 x Active-Wedge #4 (Green), 20 x Active-Wedge #5 (Black).

**COMPLETE KIT WITH ULTRA-WRAP BANDS (209022)**

- Contains: 1 x Starter Kit (see above for full list of contents), 1 x Punch Pliers, 1 x Ring Forceps.

- RING FORCEPS (209009)**
- PUNCH PLIERS (209013)**
- MATRIX FORCEPS (209024)**

**ACTIVE-WEDGE INTRO KIT (209000)**

- Contains: 40 x each size wedge (#1, #2, #3, #4, #5)

**ACTIVE-WEDGE REFILL 100-PACKS**

- #1 (209001)
- #2 (209002)
- #3 (209003)
- #4 (209004)
- #5 (209005)

**RING 2-PACK REFILLS**

- Pre-molar (209007)
- Molar (209008)
- Assorted (209006)

**ULTRA-WRAP MATRIX BAND 100-PACK REFILLS**

- 4.5mm (209018)
- 5.5mm (209019)
- 6.5mm (209020)

**ULTRA-WRAP MATRIX BAND TRIAL KIT (209023)**

To view the latest news, education and more, visit:

[www.clinicianschoice.com](http://www.clinicianschoice.com)

